Sperm Preparation Medium

Product No.:
1069
1070
Languages:
bg cs da de el
en es et fr hr
hu is it kk lt
lv mk nl no pt
ro ru sk sl sv
tr uk
Customer Service:

E-mail: customer.service@origio.com

Tel.: +45 46 79 02 02 Fax: +45 46 79 03 02

ORIGIO a/s

Knardrupvej 2, DK-2760 Måløv, Denmark

www.origio.com

Tel: +45 46 79 02 00 · Fax: +45 46 79 03 00

a CooperSurgical Company

Ingredientia
1069 Sperm Preparation Medium
Albumini humani solutio; Aqua; CaCl₂; Gentamicinum; Glucosum; HEPES; KCl; MgSO₄; NaCl; NaHCO₃; NaH₂PO₄; Pyruvas; SSR.
1070 Sperm Preparation Medium with phenol red
Albumini humani solutio; Aqua; CaCl₂; Gentamicinum; Glucosum; HEPES; KCl; MgSO₄; NaCl; NaHCO₃; NaH₂PO₄; Phenolsulfonphthaleinum; Pyruvas; SSR.

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- Soon after collection, the semen sample is thoroughly mixed (i.e. repeated tilting for 20 minutes at room temperature). If the sample does not liquefy, you may need to pass it through a narrow pipette and/or mix it with a small amount of Sperm Preparation Medium.
- After the mixing process is complete, sperm concentration and motility should be assessed under the microscope to confirm the method of washing.
- Carefully layer 0.5-1 ml of the liquefied semen in a tube underneath 1-2 ml of pre-equilibrated Sperm Preparation Medium.
- If possible, place the tubes at an angle to increase the interface between the semen sample and the Sperm Preparation Medium. This is done in order to increase recovery of the most motile spermatozoa as they migrate into the medium. Place the rack in a 5-6% CO₂ environment at 37°C for 30-60 minutes depending on the semen quality. *The swim-up can also be performed at room temperature, in which case the caps of the tubes are tightened to keep the pH of the medium stable.*
- After swim-up, the upper 0.2-1 ml is aspirated and assessed for sperm concentration and motility. If the sperm count is too low, the next 0.5 ml is included as well. Pool the aspirates together.
- During the aspiration, care should be taken not to disturb the interface between the semen sample and the media.
- If further concentration of the aspirated sperm medium is needed, add 5 ml of Sperm Preparation Medium, mix and centrifuge at 400 g for 10 minutes.

- Aspirate the supernatant and re-suspend the remaining pellet in a suitable volume of pre-equilibrated Sperm Preparation Medium. *When the caps of the tubes are tightened, the prepared semen can be kept at room temperature (20-25°C) for up to 1 hour prior to fertilisation. It is recommended that the sperm sample be wrapped in aluminium foil. Alternatively, the unwrapped sperm sample can be stored in a 5-6% CO₂ environment at 37°C.*

en - english

For washing of spermatozoa and isolation of motile viable spermatozoa by swim-up method.

This product is for ART treatment, whether the cause of infertility is male or female. The product should only be used by professionals trained in ART treatment.

Products

1069 Sperm Preparation Medium
1070 Sperm Preparation Medium with phenol red

Contains

Human albumin solution (HAS)
Recombinant human insulin
Gentamicin sulphate 10 µg/ml

Quality control testing

Sterility tested (Ph.Eur., USP)
Osmolality tested (Ph.Eur., USP)
pH tested (Ph.Eur., USP)
Endotoxin tested ≤ 0.1 EU/ml (Ph.Eur., USP)
HSA analysis (Ph.Eur., USP)
Sperm survival tested
Note: The results of each batch are stated on a Certificate of Analysis, which is available at www.origio.com.

Storage instructions and stability

The products are aseptically processed and supplied sterile.
Store in original container at 2-8°C, protected from light.
Do not freeze.

Discard excess (unused) media following warming.
The product is to be used within 7 days after opening.
When stored as directed by the manufacturer the product is stable until the expiry date shown on the label.

Precautions and warnings

Do not use the product if:

- Product packaging appears damaged or if the seal is broken.
- Expiry date has been exceeded.
- The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.

Caution: All blood products should be treated as potentially infectious. The source material used to manufacture this product was tested and found non-reactive for HBsAg and negative for Anti-HIV-1/-2, HIV-1, HBV, and HCV. Furthermore, source material has been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.

Note: Please note the need for traceability of this product. In addition, national legal requirements for this field may exist in your country.

Note: Only to be used in combination with other devices intended for the particular purpose.

Note: Dispose of the device in accordance with local regulations for disposal of medical devices.

Instructions for use

- Soon after collection, the semen sample is thoroughly mixed (i.e. repeated tilting for 20 minutes at room temperature). If the sample does not liquefy, you may need to pass it through a narrow pipette and/or mix it with a small amount of Sperm Preparation Medium.
- After the mixing process is complete, sperm concentration and motility should be assessed under the microscope to confirm the method of washing.
- Carefully layer 0.5-1 ml of the liquefied semen in a tube underneath 1-2 ml of pre-equilibrated Sperm Preparation Medium.
- If possible, place the tubes at an angle to increase the interface between the semen sample and the Sperm Preparation Medium. This is done in order to increase recovery of the most motile spermatozoa as they migrate into the medium. Place the rack in a 5-6% CO₂ environment at 37°C for 30-60 minutes depending on the semen quality. *The swim-up can also be performed at room temperature, in which case the caps of the tubes are tightened to keep the pH of the medium stable.*
- After swim-up, the upper 0.2-1 ml is aspirated and assessed for sperm concentration and motility. If the sperm count is too low, the next 0.5 ml is included as well. Pool the aspirates together.
- During the aspiration, care should be taken not to disturb the interface between the semen sample and the media.
- If further concentration of the aspirated sperm medium is needed, add 5 ml of Sperm Preparation Medium, mix and centrifuge at 400 g for 10 minutes.
- Aspirate the supernatant and re-suspend the remaining pellet in a suitable volume of pre-equilibrated Sperm Preparation Medium. *When the caps of the tubes are tightened, the prepared semen can be kept at room temperature (20-25°C) for up to 1 hour prior to fertilisation. It is recommended that the sperm sample be wrapped in aluminium foil. Alternatively, the unwrapped sperm sample can be stored in a 5-6% CO₂ environment at 37°C.*

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